



FICHA TÉCNICA COMERCIAL

Código: FT- DS - 003
Fecha Creación: Octubre 24 de 2019
Versión: 01
Fecha Modificación:

Nombre: Dama Transparente Rodilla Baja Compresión 8-15 mmHg

Ref: 1113

FOTOGRAFIA



DETALLES

Baja compresión 8-15 mmHg.

INDICACIONES

- Profilácticas.
- Telangiectasias.
- Varices primarias.
- Dolor.
- Cansancio.
- Inflamación.

BENEFICIOS

- Compresión graduada decreciente que favorece el retorno venoso.
- Banda elástica de ajuste perfecto.
- Pie con plantilla que permite la transpiración.

RECOMENDACIONES DE USO

Recomendadas por los médicos para aliviar el cansancio, pesadez y calambres.

COMPOSICIÓN

Composicion: 80% Poliamida 20% Elastano

DATOS ESPECÍFICOS

Color: Negro, Sahara, Vison y Beige.
Tallas: S, M, L y XL



FICHA TÉCNICA COMERCIAL

Código: FT- DS - 003
Fecha Creación: Octubre 24 de 2019
Versión: 01
Fecha Modificación:

Nombre: Dama Transparente Rodilla Baja Compresión 8-15 mmHg

Ref: 1113

CRITERIOS DE CALIDAD

- Conservar igualdad de tonos en prenda.
- El tejido no debe presentar defectos como huecos, marras y demás que afecten la calidad del producto.
- Debe visualizarse la prenda limpia, sin hebras fijas ni sueltas.
- Las costuras no deben presentar saltos, tensiones, destensiones.
- Costura de puntera PLANA.

RECOMENDACIONES DE LAVADO

- Lavar a mano. Temperatura máxima 40° C / Hand wash maximum temperature 40°C
- No usar blanqueador / Do not bleach
- No secar en máquina / Do not machine dry
- No dejar en remojo / Do not leave to soak
- Secado extendido a la sombra / Dry flat in shade
- No planchar / Do not iron
- No limpieza en seco / Do not dry clean
- No limpieza en húmedo profesional / Do not professionally wet clean

CÓDIGOS

Item	Nombre	Código EAN-13
1113p0101	Dama Trans. Rodilla Com. Baja 8-15 S Sahara	7707670405533
1113p0103	Dama Trans. Rodilla Com. Baja 8-15 S Vison	7707670403324
1113p0104	Dama Trans. Rodilla Com. Baja 8-15 S Negro	7707670409296
1113p0108	Dama Trans. Rodilla Com. Baja 8-15 S Beige	7707670404215
1113p0201	Dama Trans. Rodilla Com. Baja 8-15 M Sahara	7707670408404
1113p0203	Dama Trans. Rodilla Com. Baja 8-15 M Vison	7707670408176
1113p0204	Dama Trans. Rodilla Com. Baja 8-15 M Negro	7707670400590
1113p0208	Dama Trans. Rodilla Com. Baja 8-15 M Beige	7707670400392
1113p0301	Dama Trans. Rodilla Com. Baja 8-15 L Sahara	7707670406431
1113p0303	Dama Trans. Rodilla Com. Baja 8-15 L Vison	7707670403454
1113p0304	Dama Trans. Rodilla Com. Baja 8-15 L Negro	7707670400477
1113p0308	Dama Trans. Rodilla Com. Baja 8-15 L Beige	7707670406110
1113p0401	Dama Trans. Rodilla Com. Baja 8-15 XL Sahara	7707670400446
1113p0403	Dama Trans. Rodilla Com. Baja 8-15 XL Vison	7707670405182
1113p0404	Dama Trans. Rodilla Com. Baja 8-15 XL Negro	7707670404468
1113p0408	Dama Trans. Rodilla Com. Baja 8-15 XL Beige	7707670404406



FICHA TÉCNICA COMERCIAL

Código: FT- DS - 003
Fecha Creación: Octubre 24 de 2019
Versión: 01
Fecha Modificación:

Nombre: Dama Transparente Rodilla Baja Compresión 8-15 mmHg

Ref: 1113

MATERIAL EMPAQUE

Medi Varic®

MEDIAS MEDICADAS - MEDICAL STOCKINGS

COMO COLOCAR LAS MEDIAS PANTALÓN
Y LOS CALCETINES COMPRESIÓN GRADUADA

Antes de Colocarlas / Before putting on:

- Recuéstate y eleva las piernas a 45° contra una pared o superficie elevada durante 5 minutos.
- Lie down and elevate your legs at 45 degrees against a wall or raised surface for 5 minutes.
- Haga un vaciamiento venoso presionando con ambas manos uniformemente desde el tobillo hasta el muslo. (Repita este ejercicio tres veces).
- Make an emptying venous pressure with both hands evenly from ankle to thigh (Repeat this exercise three times).



Manufacturado por / Made by: Medi Varic Productos SAS
www.medivaric.com.co
Línea Nacional de Atención: 01 4 30 30 30 - Bogotá, D.C., Colombia
R.S. INVIMA 2011DM-000794



- Como colocar / How to put on:**
1. Deslice la media desmenuzándose uniformemente desde el tobillo hasta el muslo acomodándose convenientemente, sin ejercer demasiada fuerza.
 2. Introdúzcala en la pierna de los dedos y deslice la media hacia el talón.
 3. Coloque la media hacia el gemelo elevando el talón de manera homogénea.
 4. Deslice la media hacia el muslo evitando la zona de la rodilla.
 5. Verifique que piñeta, talón, piñeta y elasticidad queden en su lugar.
 6. Check toe, heel and elasticity stay in place.
 7. Slide the media to locate the elastic below the knee.
 8. Slide the media toward the knee avoiding the knee.
 9. Slide the media toward the heel avoiding the heel.
 10. Slide the media toward the heel avoiding the heel.
 11. Slide the media toward the heel avoiding the heel.
 12. Slide the media toward the heel avoiding the heel.
 13. Slide the media toward the heel avoiding the heel.
 14. Slide the media toward the heel avoiding the heel.
 15. Slide the media toward the heel avoiding the heel.
 16. Slide the media toward the heel avoiding the heel.
 17. Slide the media toward the heel avoiding the heel.
 18. Slide the media toward the heel avoiding the heel.
 19. Slide the media toward the heel avoiding the heel.
 20. Slide the media toward the heel avoiding the heel.
 21. Slide the media toward the heel avoiding the heel.
 22. Slide the media toward the heel avoiding the heel.
 23. Slide the media toward the heel avoiding the heel.
 24. Slide the media toward the heel avoiding the heel.
 25. Slide the media toward the heel avoiding the heel.
 26. Slide the media toward the heel avoiding the heel.
 27. Slide the media toward the heel avoiding the heel.
 28. Slide the media toward the heel avoiding the heel.
 29. Slide the media toward the heel avoiding the heel.
 30. Slide the media toward the heel avoiding the heel.
 31. Slide the media toward the heel avoiding the heel.
 32. Slide the media toward the heel avoiding the heel.
 33. Slide the media toward the heel avoiding the heel.
 34. Slide the media toward the heel avoiding the heel.
 35. Slide the media toward the heel avoiding the heel.
 36. Slide the media toward the heel avoiding the heel.
 37. Slide the media toward the heel avoiding the heel.
 38. Slide the media toward the heel avoiding the heel.
 39. Slide the media toward the heel avoiding the heel.
 40. Slide the media toward the heel avoiding the heel.
 41. Slide the media toward the heel avoiding the heel.
 42. Slide the media toward the heel avoiding the heel.
 43. Slide the media toward the heel avoiding the heel.
 44. Slide the media toward the heel avoiding the heel.
 45. Slide the media toward the heel avoiding the heel.
 46. Slide the media toward the heel avoiding the heel.
 47. Slide the media toward the heel avoiding the heel.
 48. Slide the media toward the heel avoiding the heel.
 49. Slide the media toward the heel avoiding the heel.
 50. Slide the media toward the heel avoiding the heel.
 51. Slide the media toward the heel avoiding the heel.
 52. Slide the media toward the heel avoiding the heel.
 53. Slide the media toward the heel avoiding the heel.
 54. Slide the media toward the heel avoiding the heel.
 55. Slide the media toward the heel avoiding the heel.
 56. Slide the media toward the heel avoiding the heel.
 57. Slide the media toward the heel avoiding the heel.
 58. Slide the media toward the heel avoiding the heel.
 59. Slide the media toward the heel avoiding the heel.
 60. Slide the media toward the heel avoiding the heel.
 61. Slide the media toward the heel avoiding the heel.
 62. Slide the media toward the heel avoiding the heel.
 63. Slide the media toward the heel avoiding the heel.
 64. Slide the media toward the heel avoiding the heel.
 65. Slide the media toward the heel avoiding the heel.
 66. Slide the media toward the heel avoiding the heel.
 67. Slide the media toward the heel avoiding the heel.
 68. Slide the media toward the heel avoiding the heel.
 69. Slide the media toward the heel avoiding the heel.
 70. Slide the media toward the heel avoiding the heel.
 71. Slide the media toward the heel avoiding the heel.
 72. Slide the media toward the heel avoiding the heel.
 73. Slide the media toward the heel avoiding the heel.
 74. Slide the media toward the heel avoiding the heel.
 75. Slide the media toward the heel avoiding the heel.
 76. Slide the media toward the heel avoiding the heel.
 77. Slide the media toward the heel avoiding the heel.
 78. Slide the media toward the heel avoiding the heel.
 79. Slide the media toward the heel avoiding the heel.
 80. Slide the media toward the heel avoiding the heel.
 81. Slide the media toward the heel avoiding the heel.
 82. Slide the media toward the heel avoiding the heel.
 83. Slide the media toward the heel avoiding the heel.
 84. Slide the media toward the heel avoiding the heel.
 85. Slide the media toward the heel avoiding the heel.
 86. Slide the media toward the heel avoiding the heel.
 87. Slide the media toward the heel avoiding the heel.
 88. Slide the media toward the heel avoiding the heel.
 89. Slide the media toward the heel avoiding the heel.
 90. Slide the media toward the heel avoiding the heel.
 91. Slide the media toward the heel avoiding the heel.
 92. Slide the media toward the heel avoiding the heel.
 93. Slide the media toward the heel avoiding the heel.
 94. Slide the media toward the heel avoiding the heel.
 95. Slide the media toward the heel avoiding the heel.
 96. Slide the media toward the heel avoiding the heel.
 97. Slide the media toward the heel avoiding the heel.
 98. Slide the media toward the heel avoiding the heel.
 99. Slide the media toward the heel avoiding the heel.
 100. Slide the media toward the heel avoiding the heel.



ALMA PARA MEDIA DE RODILLA
BOLSA INTERNA ALTAPACK

CAJA INDIVIDUAL

PESO CAJA IND.		
S	0,050	gr
M	0,052	gr
L	0,054	gr
XL	0,056	gr

MEDIDA CAJA IND.		
A	Altura 1	15,5
B	Altura 2	15
C	Ancho	9,5
D	Profundidad	2,8
E	Lomo	2,8

STICKER





FICHA TÉCNICA COMERCIAL


Código: FT- DS - 003
Fecha Creación: Octubre 24 de 2019
Versión: 01
Fecha Modificación:

Nombre: Dama Transparente Rodilla Baja Compresión 8-15 mmHg

Ref: 1113

MATERIAL EMPAQUE

CAJA SIX PACK PEQUEÑA

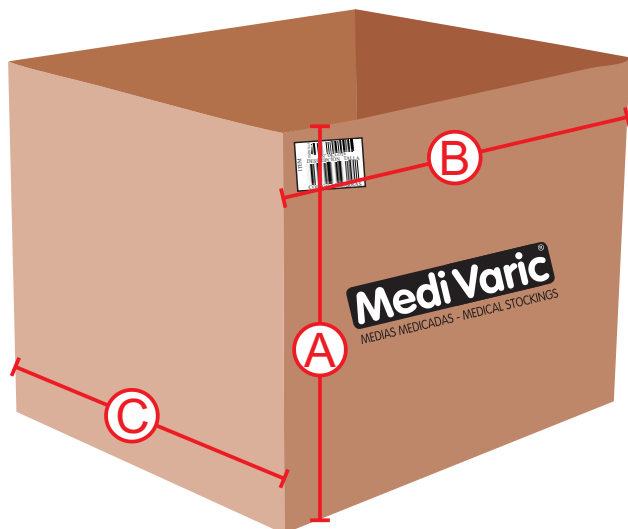

MEDIDA CAJA SIX PACK


A	Altura	18
B	Ancho	16
C	Profundidad	10

PESO CAJA SIX PACK

S	0,340	gr
M	0,352	gr
L	0,364	Kl
XL	0,376	Kl

CANTIDAD 6 UNIDADES

CAJA MASTER

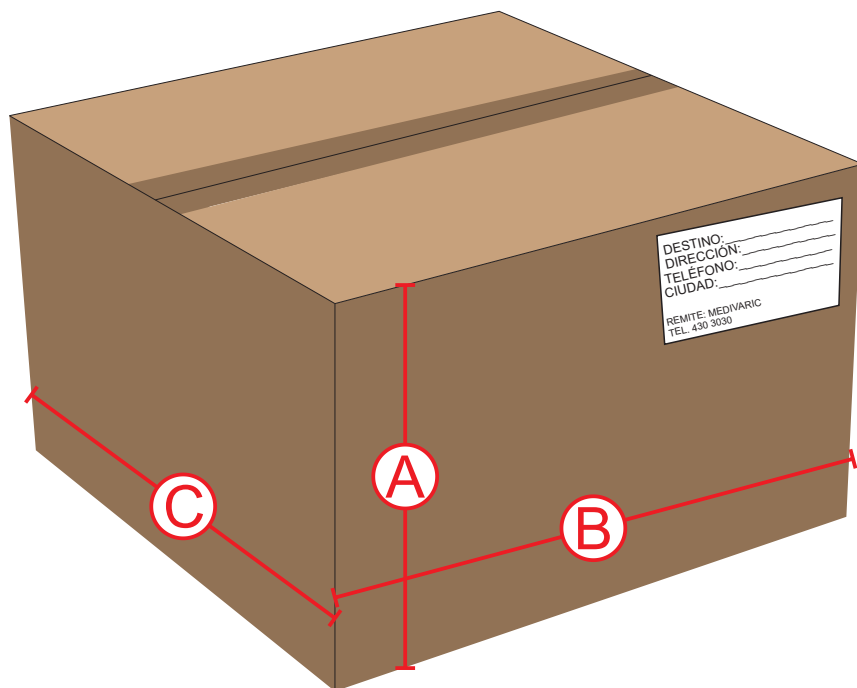

MEDIDA CAJA MASTER

A	Altura	42
B	Ancho	55
C	Profundidad	40

PESO CAJA MASTER

S	8,16	Kl
M	8,45	Kl
L	8,74	Kl
XL	9,02	Kl

CANTIDAD 144 UNIDADES